



# Dentalrapid<sup>®</sup> SD

wipes

## Alcoholic Disinfectant Wipes READY-TO-USE



### Advantages:

- ▶ Short application times
- ▶ Extremely broad spectrum of effectiveness
- ▶ **Limited Virucidal PLUS**
- ▶ All cloth-impregnating systems are tested
- ▶ 4-Field-Test
- ▶ Easy documentation

### Product type:

Ready-to-use disinfectant wipes in special lint-free nonwoven qualities (see delivery forms) with bonded fibres in various formats and application forms. All cloth impregnating systems are tested with the 4-Field Test (EN 16615) and accordingly listed as an independent product acc VAH-guidelines. The impregnating liquid Dentalrapid SD liquid, is also VAH listed.

### Type of application:

**As per the Medical Devices Law (MPL):** Alcoholic wet wipes for the prophylactic surface disinfection of alcohol-resistant medical products. Not suitable for the terminal disinfection of invasive medical products. As per the Biocidal Products Directive: Alcoholic wet wipes for the fast disinfection of alcohol-resistant surfaces. **According to Biocides Directive (if registered):** Alcoholic wipe disinfection for the disinfection of all alcohol resistant surfaces.

### Indication for use:

#### **Cloth impregnating systems with dispenser:**

##### Dentalrapid SD wipes S / M / XL

Open the top of the bag and put it in the prepared dispenser (box for S and container for M and XL). Pull on the first wipe of the roll and thread it through the star-shaped hole in the lid. Push the lid down firmly onto the container and seal it tightly. Carefully wipe the area to be disinfected and let the disinfectant take effect. Only for use on alcohol-resistant materials (e.g. not suitable for acrylic glass). After usage always close the lid carefully and tightly. Once the bag has been opened, the wipes can be used for 28 days in a suitable container.

## Documentation of cloth impregnating systems:

### Dentalrapid SD wipes M und XL

Lift the complete Lift & Stick label from the bag, stick it onto the container and fill in the opening and expiry date (+ 28 days).

### Dentalrapid SD wipes S and L

For the smallest cloth variant Dentalrapid SD wipes S there is no Lift & Stick label. Here, the opening and expiry date (+ 28 days) is entered on the label of the box. In the case of Dentalrapid SD wipes L, documentation is provided directly on the label of the flow pack.

## Preparation guidelines:

The dispenser is to be prepared with a suitable disinfectant wipe or solution, e.g. Dentalrapid SD liquid in accordance with the preparation guidelines below. During application, wear suitable gloves (e.g. made of nitrile rubber). For Dentalrapid SD wipes L this procedure is not necessary.



1. Open the dispenser
2. Disinfect the container and lid from the inside and outside (notice application time).
3. Let it dry
4. Open the new bag, put it into the container/dispenser, pull the wipe through the lid and seal it tightly.

**Spectrum of effectiveness:** Bactericidal (incl. MRSA), yeasticidal (candida albicans), tuberculo-cidal (m. terrae), "limited virucidal effect" (effective against enveloped viruses such as SARS-CoV-2, HBV, HIV, HCV), and effective against adeno-, noro- and rotaviruses (**limited virucidal PLUS**).

## Application times:

(related to the impregnating liquid):

Bacteria as per EN 13727 and EN 13697

(dirty conditions): conc. 1 min.

Candida albicans as per EN 13624 and

EN 13697 (dirty conditions): conc. 1 min.

TbB (tuberculosis) inactivation (M. terrae) as per EN

14348 (dirty conditions): conc. 1 min.

Enveloped viruses (e.g. SARS-CoV-2, HBV, HCV and HIV) as per DVV/2012

(dirty conditions): conc. 1 min.

Adeno-, noro- and rotaviruses as per EN 14476

(dirty conditions): conc. 1 min.

## Application times (related to the impregnated

wipes): Bacteria and candida albicans (as per VAH, version: 02.04.15 and EN 16615, high contamination):\* conc. 2 min.

(\*Wipe-Impregnation-System listed for 5 min. in the VAH-list.)

## 100 g of the impregnating liquid contains:





43.0 g ethanol, 8.0 g 1-propanol, 0.055 g didecyl-dimethylammoniumchloride.

**Safety notes:** H 228 flammable solid. Keep away from heat/sparks/naked flames/hot surfaces. Do not smoke. Wear protective gloves/eye protection. Store in a closed container. Flash point as per DIN 51755: approx. 25° C. Application max. 50 ml per m<sup>2</sup>. The total applied amount per room must not exceed more than 100 ml per m<sup>2</sup> of room area. Keep the containers tightly sealed.











**Notes on disposal:** Only put the packaging into the recycling system if it is empty of residue. Disposal of residual product in accordance with EAK 07 06 99. Directive 648/2004/EC: < 5 % cationic surfactants, fragrances.



## Cloth Quality and sizes:

Product Type	Size	Quality
Dentalrapid SD wipes	 12 x 20 cm	23 g / m²
Dentalrapid SD wipes	 23 x 15 cm	50 g / m²
Dentalrapid SD wipes	 20 x 30 cm	50 g / m²
Dentalrapid SD wipes	 28 x 30 cm	50 g / m²

## Delivery forms:

Product Type	Packing Unit	REF
Dentalrapid SD wipes, (lemon), refill bag, 100 tissues	 12	71510100
Dentalrapid SD wipes, (lemon) 2 rolls with 240 tissues	 1	71510240
Dentalrapid SD wipes, (lemon) Flowpack, 80 Tücher	 6	71510080
Dentalrapid SD wipes, (neutral) 2 rolls with 90 tissues	 1	71500090
Dentalrapid SD wipes, (lemon) 2 rolls with 90 tissues	  1	71510090
Dentalrapid SD wipes, (fruit) 2 rolls with 90 tissues	  1	71520090
Dentalrapid SD wipes, (flower) 2 rolls with 90 tissues	  1	71530090

## Particularly suitable accessory:

Product:	Packing unit	REF
Universal-dispenser Dispenser cube, empty	1	701005012
Universal-dispenser Dispenser box, empty	1	701000100



**Shelf life:** 3½ Years from the date of manufacture.

## Contact:

Further information about reports / test results and about the 360° hygiene program is available on request.

Tel.: +49 (0) 2266 4742 - 0

E-Mail: [info@mueller-omicron.de](mailto:info@mueller-omicron.de)

## A Quality Brand from

Müller-Omicron GmbH & Co. KG

Schlosserstr. 1

D-51789 Lindlar

Tel.: +49 (0) 2266 4742-0

Fax: +49 (0) 2266 4742-23

[www.mueller-omicron.de](http://www.mueller-omicron.de)

[www.mueller-omicron.com](http://www.mueller-omicron.com)