

Product Information **SURFACE** Disinfection



**Alcohol-free Disinfectant Wipes
READY-TO-USE**

Advantages:

- ▶ **alcohol-free** and pH-neutral
- ▶ Large spectrum of effectiveness
- ▶ Impregnating wipe system tested
- ▶ Four-field test
- ▶ Acrylic glass suitable

Product type:

Alcohol-free, ready-to-use disinfectant wipes in special lint-free non-woven quality (see delivery forms) with consolidated fibers for cleaning and disinfection of non-invasive surfaces. The wipe impregnation system has been tested with the four-field test (EN 16615) and is accordingly VAH-listed as an independent product. The impregnating liquid used, Dentalrapid soft SD liquid, is also VAH-listed.

Type of application: As per the Medical Devices Law (MPG): Alcohol-free wet wipes for the prophylactic surface disinfection of non-invasive medical products. Particularly suitable for use on alcohol-sensitive surfaces like acrylic glass.

Indication for use:

Remove the stick & lift label from the bag, stick it onto the intended dispenser tub and enter the use by date etc. Open the bag at the top and place it in the intended dispenser tub. Pull on the first wipe of the roll and thread it through the lock in the lid clip. Push the lid down onto the tub and close it. Carefully wipe the area to be disinfected and let the disinfectant take effect. Always close the container.

After opening the bag, the wipes can be used for 28 days in the dispenser tub. Before using the product on large areas of sensitive materials like artificial leather, first test the material tolerance on a small, hidden area. To prevent streaks forming, the treated areas should be wiped dry at regular intervals with a damp (with water) disposable cloth. The disposal tub is to be prepared in accordance with the adjacent preparation guidelines, using a suitable disinfection cloth. During application, suitable gloves (e.g. made of nitrile rubber) are to be worn.

Spectrum of effectiveness:

Bactericidal (incl. MRSA) as per VAH, EN 13727, EN 13697, EN 1276, EN 16615; levurocidal (candida albicans) as per VAH, EN 13624, EN 13697, EN 1650, EN 16615; "limited virucidal effect" (effective against enveloped viruses such as SARS-CoV-2, HBV, HIV, HCV); rotaviruses as per RKI/DVV; noroviruses as per EN 14476.

Application times

(related to the impregnating fluid):

Bacteria and Candida albicans as per EN 13727, EN 13624, EN 1276, EN 1650

(dirty conditions): conc. 1 min.

Bacteria and Candida albicans as per EN 13697 (dirty conditions): conc. 2 min.

Enveloped viruses (e.g. SARS-CoV-2, HBV, HCV and HIV) as per RKI/DVV, dirty conditions: conc. 1 min.

Rotaviruses as per EN 14476 (clean conditions): conc. 1 min.

Noroviruses as per EN 14476, (clean conditions): conc. 30 min.

Polyomavirus SV40 (RKI/DVV) with pollution): conc. 30 min.


Effectiveness of the impregnated wipes as per EN 16615 and VAH (dirty conditions): conc. 1 min.

Documentation of cloth impregnating systems:

Lift the complete Lift & Stick label from the bag, stick it onto the container and fill in the opening and expiry date (+ 28 days).

Preparation guidelines:

The dispenser is to be prepared with a suitable disinfectant wipe or solution, e.g. Dentalrapid SD liquid in accordance with the preparation guidelines below. During application, wear suitable gloves (e.g. made of nitrile rubber). For Dentalrapid SD wipes L this procedure is not necessary.

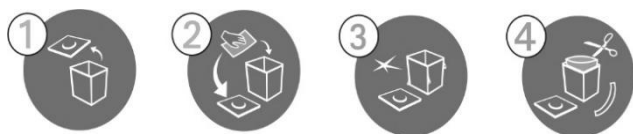

1. Open the dispenser
2. Disinfect the container and lid from the inside and outside (notice application time).
3. Let it dry
4. Open the new bag, put it into the container/dispenser, pull the wipe through the lid and seal it tightly.

Contents:

90 units of PES/VIS mixed material; 28cm x 30cm, impregnated with Dentalrapid soft SD liquid

100 g of the impregnating fluid contains:

0.24g Benzyl-C12-16-alkyldimethylammonium chloride, 0.24g N,Ndidecyl-N,N-dimethyl ammonium chloride, 0.24g C12-14-akyl[(ethylphenyl)methyl] dimethyl ammonium chloride

100 g of the impregnating fluid contains:

0.24g Benzyl-C12-16-alkyldimethylammonium chloride, 0.24g N,Ndidecyl-N,N-dimethyl ammonium chloride, 0.24g C12-14-akyl[(ethylphenyl)methyl] dimethyl ammonium chloride

Notes on disposal: Only put the packaging into the recycling system if it is empty of residue. Dispose of residual product in accordance with EAK 07 06 99. 648/2004/EC: < 5% cationic surfactants, fragrance.

Delivery forms:

Product type	Packing Unit	REF
Dentalrapid soft SD wipes XL 2 rolls with 90 tissues	1	71590090

Cloth Quality and sizes:

Product	Size	Quality
Dentalrapid soft SD wipes XL	28 x 30 cm	50 g / m ²

Particularly suitable Accessory:

Produkt:	VE	REF
Universal-dispenser, Dispenser cube, empty	1	701005012



Shelf life: 3 ½ Years from the date of manufacture

Contact:

Further information about reports / test results and about the 360° hygiene program is available on request. Tel.: +49 (0) 2266 4742 - 0
E-Mail: info@mueller-omicron.de

A Quality Brand from

Müller-Omicron GmbH & Co. KG

Schlosserstr. 1

D-51789 Lindlar

Tel.: +49 (0) 2266 4742-0

Fax: +49 (0) 2266 4742-23

www.mueller-omicron.de

www.mueller-omicron.com