The Cleaner CONCENTRATE

Advantages:

- ▶ Low-foam cleaning concentrate
- ▶ Very good cleaning performance
- ▶ Good material compatibility
- ▶ Preventive effect against constipation

Type of application:

Foam-inhibited cleaning concentrate for the cleaning of dental suction systems incl. their drainage conduits. Regular usage prevents the formation of deposits and blockages or dissolves already existing deposits.

Indication for use:

The special cleaner is used as a 10 % solution for problem cases (100ml/l water). The application time is 1-2 hours. For regular use, the cleaner is applied as a 5 % solution (50ml/l water) one or two times a week with a application time of 30 min. 2 litres of the usage solution are sucked through the suction tubes and rinsed with one litre of water through the system after the application time.

Composition:

- < 5 % amphoteric surfactants
- < 5 % phosphonates

Ingredients: (100 g of the solution contain):
7.5g Dimethyldioctylammoniumchloride, 0.6g Benzyldimethyldodecylammoniumchloride

Chemical-physical data:

- ▶ Appearance: transparent
- ▶ Density: 1.2 g / cm³
- ▶ pH value at 20° C: < 2

Application concentration:

Solution for use	5,0 %	10,0 %
1 Litre	50 ml	100 ml
2 Litre	100 ml	200 ml
3 Litre	150 ml	300 ml
5 Litre	250 ml	500 ml
10 Litre	500 ml	1000 ml

Safety notes:

H319 - Causes serious eye irritation. Wash thoroughly after handling. Wear protective gloves and eye protection. If in EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice / attention 648/2004/EG: <5 % amphoteric surfactants, <5 % Phosphonate

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Müller-Omicron GmbH & Co. KG